



### Multiple Placement Options

With two placement options available to you, bonding MTM<sup>®</sup> No-Trace<sup>™</sup> System brackets and archwires is a simple process. Remarkably similar to a typical labial procedure, placement of MTM<sup>®</sup> No-Trace<sup>™</sup> System typically requires about 30 minutes of chair time to complete.

### Direct Placement

Direct placement of MTM<sup>®</sup> No-Trace<sup>™</sup> System can be completed in 3 easy steps.

- 1 Prepare the teeth as you would for normal labial bonding.
- 2 Apply adhesive and position the brackets 1-2 millimeters from the incisal edge and cure them in place.
- 3 Once the brackets are completely cured, the archwire can be pre-attached and inserted into the self-ligating MTM<sup>®</sup> No-Trace<sup>™</sup> System brackets. You can use nothing more than thumb pressure to close the clip and secure the wire in the same manner as labial.

### Indirect Placement

The indirect bonding service is a convenient, easy to use option providing pre-positioned brackets in an indirect transfer tray. You also have the option of having the MTM<sup>®</sup> Service Center preset the initial Secondary archwire in the brackets with steps within 10 minutes.

- 1 Submit an Impression and case Rx form to the MTM Service Center.
- 2 Models are fabricated, brackets are etched and high-resolution images of the case are posted on-line for doctor review and approval.
- 3 Transfer trays are fabricated from a special polymer that demonstrates high flexibility while maintaining stability.
- 4 Custom bases are created that conform to the contours of each tooth so bonding to the enamel can be accomplished with a thin coating of the adhesive provided.
- 5 An MTM<sup>®</sup> No-Trace<sup>™</sup> System Patient Kit is then shipped to you to complete your case.

### Align with the Expert

"One of the most difficult things for any lingual orthodontist to do is to tie in the archwire. Most lingual orthodontists will spend an hour to an hour and ten minutes to do one, or maybe even two arch changes, while with the advent of self-ligating's pretty simple. With MTM<sup>®</sup> No-Trace<sup>™</sup> System you put it in a very, very good way. You come out with a great practice bubble, it's great practice bubble, it's easy to do and they're in minutes. It's great and you can't do it any other way. It's a great practice bubble, it's easy to do and just about any orthodontist can do it and feel really, really comfortable that he's doing a good job."

Dr. Ron Roncone, Peabody Orthodontics, Peabody, MA, CA

### Dr. Roncone - MTM<sup>®</sup> No-Trace<sup>™</sup> Cases



### The Straightest Routes to Getting Started

MTM<sup>®</sup> Welcome Kits are a fast and convenient way to attract new patients and grow your practice.

### Included in Your Kit

	MTM <sup>®</sup> No-Trace <sup>™</sup> System Direct Bond Welcome Kit	MTM <sup>®</sup> No-Trace <sup>™</sup> System Indirect Bond Welcome Kit
(1) 17" Digital Frame	X	X
MTM Patient Brochures	50	100
(1) MTM System Guide	X	X
OR FREE Indirect Transfer Tray Vouchers	X	X
(10) MTM No-Trace Brackets UL-L	X	X
(1) MTM No-Trace Consultation Model	X	X
(1) Recosone Bonding Instrument Set	X	X
(1) Mo's Sucker Opener	X	X
(1) Bracket Placement Instrument	X	X
OR Sensibly Multicolor Archwires (L12)	X	X
(10) Resilite Blue Titanium Archwires	X	X
(1) Lingual Arch Form Template	X	X
OR Compatible Split Steps	X	X
(1) Neo Bond & Adhesive Kit	X	X
(20) Patient DNDs	X	X



For more information contact your DENTSPLY GAC Representative or call 888.895.MTM (6889)

©2010 DENTSPLY GAC International. All rights reserved. 120-000-03 Rev. 1/2011



# Get It Straight

## MTM<sup>®</sup> No-Trace<sup>™</sup> System The Brace Without a Trace



- ✓ Practice Building Opportunity
- ✓ Easy to Incorporate
- ✓ Highly Esthetic



MinorToothMovement.com

888.895.MTM (6889)

## Join the Movement

Introducing "Micro Tooth Movement"™ (MTM), a new treatment category from DENTEGLY International designed to correct the most common minor anterior malalignments seen in adult patients. The MTM™ No•Trace™ System is an easy-to-implement treatment option within this new category that offers you a unique opportunity to expand your practice into an already mature—and highly profitable—market segment.

## MTM™ No•Trace™ System—The Brace Without a Trace

MTM™ No•Trace™ System is the all-but-undetectable lingual treatment that works behind the scenes for patients requiring up to 100% correction in as little as 12-24 weeks. And because there's no action required by the patient, compliance is nearly effortless.

MTM™ No•Trace™ System utilizes small, easy-to-bond, self-ligating, lingual brackets that virtually eliminate the need for archwires. The MTM™ No•Trace™ System archwires typically takes no more than 30 minutes when bonding directly - a convenient indirect option is also available.

- ✓ Fixed Low-Profile, Lingual Bracket Option
- ✓ Ease of Self-Ligation
- ✓ Place Direct or Indirect
- ✓ New Treatment Option = New Patients

## The Advantages of MTM™ No•Trace™ System

- Fast, easy placement with direct or optional indirect bonding.
- Rigid wire engagement with self-ligating brackets.
- Eliminates compliance issues that can undermine a successful clinical outcome.
- Minimal wire changes that can consume chair time and undermine profitability.
- Small, low profile brackets for maximum patient comfort.



## Patient Selection

The MTM™ No•Trace™ System is the ideal choice for teens and adults who desire a quick and efficient solution to cosmetic and/or relapse needs.

Selecting the right cases is a critical component of a successful program. The MTM™ System is designed to treat the mild-to-moderate minor anterior alignment cases pictured below.

\* Patients should have all their permanent teeth.



## Indications For Use

1	Space Closure 0 - 4mm	
2	Crowding per Arch Overlap b/w any 2 teeth 0.5mm	
3	Mild-Line Correction per Arch 0 - 2mm	
4	Tipping per Tooth (Mesio/Lingual) No rotations (Meso/Lingual) 0 - 3mm	
5	Rotation per Anterior Teeth No rotations	
6	Intrusion per Tooth Extrusion per Tooth 0 - 2mm 0 - 4mm	

## MTM™ No•Trace™ Contraindications

- Primary Teeth
- Uninterrupted Permanent Teeth
- Active Periodontal Disease
- Class II and III Orthodontic Cases - for esthetic result only
- Closing Open Bites

## Archwires

Treatment with MTM™ No•Trace™ System usually requires the use of only one or two archwires throughout the entire case. If the indirect option is chosen, the MTM™ Service Center can include the initial Sentralloy archwire with your transfer tray. Simply indicate the desired wire and the wire will be present in the brackets upon delivery. The following are typical treatment protocols for the suggested wires.

<b>012 Multibracket Super Elastic NITI</b> Small, Medium, Large, X-Large Arch Form		
<b>Treatment Protocol</b>	Usually used as an initial archwire (in most cases). In slightly crowded cases this may be the only wire used.	
<b>Duration</b>	Clinical application will vary.	
<b>014 Sentralloy Super Elastic NITI</b> Small, Medium, Large, X-Large Arch Form		
<b>Treatment Protocol</b>	May be a first or second archwire.	
<b>Duration</b>	Clinical application will vary.	
<b>016 Resche Universal Arch Form</b> Rich Titanium		
<b>Treatment Protocol</b>	This is a Beta Titanium wire which can be bent into a mushroom arch and additionally have steps up or step down bends placed in it.	
<b>Optional Treatment Protocol</b>	Beta Titanium wire can be used with a closing loop in space closure situations.	
<b>Duration</b>	Clinical application will vary.	

## Accessories

<b>Crimpable Stops</b>		Often used to advance a wire so that teeth may be moved forward when desired. The stops are secured to the wire by using a dental end cutting plier and inserting it into space on the stop.
<b>Sentralloy Stop Wauing™ Open Coil Springs</b>		Often used to open space between crowded teeth so that brackets can be properly placed anterior to make room so that wires can be properly engaged into brackets.